USER GUIDE FOR DISPLAYED MESSAGES

Code no.	Description and User Instruction	Learn More
62: The right temperature for building up pressure for sterilization has not been reached within 7.5 min.	Increase the "Ambient sensitivity" level settings → Pour demineralized or distilled water (below 3.0µS/cm) into the water tank if the water level is too low → If the autoclave is placed in a cold spot remove it to surroundings at room temperature → Press C to reset and then restart the cycle	User manual, Sec.4
64: The safety system has been activated	Press C → Restart the autoclave.	User manual, Sec.4
67: The time of sterilization has been less than 3 min.	The instruments should be sterilized again	User manual, Sec.4
70: The pressure level in the chamber does not decrease fast enough after pressure test	Empty the chamber for water → Clean or change the filter → Check that the drain tube is not blocked or broken → Empty the drain tank if full	User manual, Sec.4 YouTube Video 70
71: The autoclave blocks due to use of bad water	Empty the water tank if necessary → Pour in fresh demineralized or distilled water (below 3.0µS/cm)	User manual, Sec.4 YouTube Video 71
72: The closing system was blocked as the lid was on its way up	Remove possible blocking of the lid	User manual, Sec.4
73: The sterilization temperature has not been reached fast enough during the back-flush	Pour demineralized or distilled water (below 3.0µS/cm) into the water tank if the water level is too low → Press C to reset and then restart the cycle	User manual, Sec.4
74: The right temperature for building up pressure for sterilization has not been reached fast enough	Pour demineralized or distilled water (below 3.0µS/cm) into the water tank if the water level is too low → Increase the "Ambient sensitivity" level settings → Press C to reset and then restart the cycle	User manual, Sec.4
75: The right sterilization temperature has not been reached fast enough	Pour demineralized or distilled water (below 3.0µS/cm) into the water tank if the water level is too low → Increase the "Ambient sensitivity" level settings → Press C to reset and then restart the cycle	User manual, Sec.4
81: Low level in either the water tank or the oil bottle	Pour demineralized or distilled water (below 3.0µS/cm) into the water tank or change the oil bottle	User manual, Sec.4 YouTube Video 81
84: The sterilization temperature has not been maintained during sterilization	Pour demineralized or distilled water (below 3.0µS/cm) into the water tank if needed	User manual, Sec.4 YouTube Video 84
86: The safety ring has been activated	Press C to reset the autoclave and then restart the machine → Make sure that the handle is correctly placed in the holder	User manual, Sec.4 YouTube Video 86
88: The steam pressure is too low during sterilization	Pour demineralized or distilled water (below 3.0µS/cm) into the water tank in case of low water level → Press C to reset the autoclave and then restart the cycle	User manual, Sec.4
89: The steam pressure is too high during sterilization	Check that the drain tube is not blocked, broken or bent → Empty the drain tank if full → Press C to reset the autoclave and then restart the cycle	User manual, Sec.4
90: The pressure during pressure test was too low	Check that your compressor is turned on → Check the gasket on the lid → Remove potential dirt → Make sure that the lid is placed correctly in the lid holder Check if the O-ring for the filter house is mounted correctly → Change the O-ring if damaged → Mount the lid for the filter house correctly → Replace the gasket on the lid if it is broken	User manual, Sec.4 YouTube Video 90
91: The pressure fell during the pressure test	Check the gasket on the lid → Remove potential dirt → Make sure that the lid is placed correctly in the lid holder Check if the O-ring for the filter house is mounted correctly → Change the O-ring if damaged → Make sure that the lid is placed correctly in the holder → Mount the lid for filter house correctly → Replace the gasket on the lid if it is broken	User manual, Sec.4 YouTube Video 91
92: The power supply to the autoclave has been cut out, or the C button has been pressed during the cycle	Press C to reset the machine and then restart the cycle	User manual, Sec.4 YouTube Video 92
94: Oil consumption during standby	Disconnect power from the autoclave and wait 10 seconds → Reconnect the autoclave to the power and then restart	User manual, Sec.4
95: This error can only occur if the autoclave is connected to an automatic water supply. The autoclave has "asked" for water more than 7 times during standby	Disconnect power from the autoclave and wait 10 sec. → Reconnect the autoclave to the power and then restart → Check that water is being supplied to the autoclave → Check if the autoclave has a leakage.	User manual, Sec.4
97: No water during external wash	Press C to reset the machine → Verify that the chamber is drained. If not, empty the chamber (see section 3.5 Drain Chamber in the User manual) → Restart the cycle	User manual, Sec.4
98: Lid did not close	Press C to reset and wait until the lid is in top position → Make sure that the lid is placed correctly in the lid holder (look for things that might obstruct complete closure and remove these) → Start the machine again	User manual, Sec.4

If the above does not solve the problem or for other possible displayed messages, please refer to the user manual.

Alternatively, please **contact your authorized supplier.**

Check out tech forum videos on our YouTube channel

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