LED-Montage

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LED installation

ATTENTION! Electrostatic discharge can destroy the LED. Ground/discharge yourself, e.g. by touching a water faucet, before installing the LED.

ATTENTION! The LED is in risk class 2 according to the IEC 62471:2006 standard. The LED emits optical radiation that is potentially hazardous and may be harmful to the eyes! Potential damage to the retina from the blue light emission. Do not stare at the LED for longer periods of time while in operation.

The LED can be used instead of the halogen lamp in the following Sirona treatment centers: M1-90, M1-94, M1-96, Sirona E, C2, C2+, C3, C3+, C4, C4+, C5+, C6, C8, C8+, M1+, Profeel, Profeel+, INTEGRO, INTEGRO pro, SINIUS and TENEO. The LED can also be used for the SIROTorque L / L+ table top device.

Combined operation with the Sirona halogen lamp is possible without any further action. In the case of combined operation with lamps from third-party manufacturers, the voltage settings must be adapted for each individual instrument. The voltage range of the LED is 3.2 to 3.6V.

Remove the halogen lamp from the socket as shown in the figures opposite. ATTENTION! The halogen lamp may still be hot.

The instrument light voltage for Sirona treatment centers is set to 3.6V by default. If this value has been changed, the voltage must be reset to 3.6V. Set the light intensity to 100% (for Sirona E, C6, C8, C8+, M1+ and Profeel, set it to 90%). For further information, please refer to the Operating Instructions for your treatment center.

ATTENTION! The LED may break if you press too hard on the lens. Therefore, please use the installation tool provided to insert the LED.

To test the polarity, first insert the LED gently into the socket. In doing so, note the position of the contact surfaces. Activate the instrument and check whether the LED lights up (A). If it does not light up, the LED must be rotated by 180° (B). The notch in the LED indicates the positive pole and can be used for alignment purposes.

Now insert the LED fully into the socket using the installation tool. Then detach the installation tool from the LED (C).

If you have difficulties when installing the LED, your allocated depot will be happy to help.

Keep this slip with the Operating Instructions for the instrument with the LED light.

We cannot guarantee compatibility with external devices. In this case, please contact the manufacturer.

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