 HENRY SCHEIN®

900-9666

ELECTROGALVANIZING BATH PLUS

Baño electrolítico para pulir plus

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HS Elektrolyt plus

Bagno elettrolitico PLUS

Elektrolytisch glansbad PLUS

 Distributed by (in US only)
HENRY SCHEIN INC.
135 DURYEA ROAD
Melville, NY 11747 USA

1L e

HENRY SCHEIN U.K. HOLDINGS LTD.
MEDCARE HOUSE, CENTURION CLOSE
GILLINGHAM BUSINESS PARK
GILLINGHAM, ME8 0SB U.K.
TEL:+44 870 849 0872
www.henryscheinbrand.com

Rev.2015/02

+H65890096660R

Danger
Peligro
Danger
Gefahr
Pericolo
Gevaar

LOT

UK Area of Application:

The Electrogalvanizing Bath provides a galvanic finish for chrome-cobalt alloys. It provides an excellent depth effect and can be used in all standard polishing units (preferably polishers with moving electrodes).

Application Information:

We recommend that the metal framework be trimmed properly prior to polishing. There should especially be no remaining oxides, as these areas would darken. As a precaution, the entire metal framework should be sandblasted and cleansed with a steam cleaner. After initial sandblasting and cleansing, attach the framework to the electrode, immerse it in the electrogalvanizing bath and set the device with appropriate power and time settings. Our recommendation: 5 A for 3-5 min at 50-60°C. Please take into account that there is a faster effect when warm. As there is a subtle material abrasion during electrogalvanizing, we recommend to protect thin framework areas (for ex. brackets) with wax. Electrogalvanizing bath is hygroscopic, the container should be tightly closed after each use and kept closed at all times.

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Causes severe skin burns and eye damage. Wear protective gloves/protective clothing/eye protection/face protection. Do not breathe mist/vapours/spray. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician. IF SWALLOWED: rinse mouth. Do NOT induce vomiting. Dispose of contents/container in accordance with local/regional/national/international regulations. **Contains:** ethanediol, sulphuric acid.

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