FOR PROFESSIONAL USE ONLY

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## NON-EUGENOL TEMPORARY CEMENT

### **Description:**

**Non-Eugenol Temporary Cement** is a low viscosity, non-eugenol polycarboxylate based temporary luting and filling material. Non-Eugenol Temporary Cement is indicated for routine temporisation of provisionals. Non-Eugenol Temporary Cement will not interfere with resin bonding. Clinical indications for Non-Eugenol Temporary Cement are temporary cementation of crowns, bridges, inlays and onlays and endodontic access.

**Non-Eugenol Temporary Cement** working time: 1.5 minutes, set time: 4 minutes.

*Important Note: Polycarboxylate cement is acidic when initially placed. Patients should be anaesthetised or they could experience discomfort.*

### **Manual Mix Procedure:**

1. Remove cap. Express a small amount on pad to ensure even flow.
2. Mix conventionally with mixing pad and disposable spatula for 20 - 30 seconds.
3. Deliver to provisional.

### **Removal of Temporary Cement:**

1. Remove provisional.
2. Scrape off residual cement with appropriate hand instrument.
3. Scour preparation.
4. Wash thoroughly.

### **Precautions:**

1. Non-Eugenol Temporary Cement is water soluble until set.
2. While being stored or mixed, chemical cannot come in contact with metal otherwise polymerization process will begin.
3. Patients should be anaesthetised to prevent discomfort during cementation as polycarboxylate, and other acid based cements (e.g. glass ionomer and zinc phosphate) can cause discomfort at least until fully set.

Store at room temperature. **DO NOT REFRIGERATE.**

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