 HENRY SCHEIN®

HS-XRV

X-RAY VIEWER

NEGATOSCOPIO

NEGATOSCOPE


HS-RÖNTGENFILMBETRACHTER

NEGATOSCOPIO

RÖNTGENVIEWER

INSTRUCTIONS FOR USE
INSTRUCCIONES DE USO
MANUEL D'UTILISATION
GEBRAUCHSANWEISUNG
ISTRUZIONI PER L'USO
GEBRUIKSAANWIJZING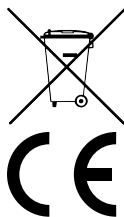

Distributed by:
Distribuido por: Vertrieb durch:
Distribué par: Distribuito da:

HENRY SCHEIN INC.
Melville, NY 11747 USA

EC REP

Henry Schein U.K. Holdings Ltd.
Gillingham ME8 0SB U.K.

Users manual for HS-XRV 30 / 38

X-Ray viewer

Table of Contents

GENERAL NOTES	0
DESCRIPTION OF THE EQUIPMENT AND ITS USE	1
TECHNICAL SPECIFICATIONS	1
IMPORTANT WARNINGS	1
INSTRUCTIONS FOR USE	1
MAINTENANCE	1

GENERAL NOTES :

Please read this booklet carefully. It contains important information about safe installation, use and maintenance. Keep this booklet in a convenient place for further reference.

- (1)** Check the general condition of the unit after removing the packaging. When in doubt do not operate the unit and contact a qualified service repair person authorised by **HENRY SCHEIN**.
- (2)** Before connection, check to see that the identification plate indicates the correct voltage for the electrical supply system. Installation must be carried out in compliance with all applicable safety standards and in accordance with the manufacturer's instructions. The manufacturer will not be held responsible for damage to objects and/or harm to people caused by improper installation. The unit is safe only when connected correctly to ground via an appropriate device in accordance with safety standards.
This fundamental requirement must be satisfied. If in doubt, request a thorough inspection of the system by a professionally qualified technician.
The manufacturer will not be held responsible for damage caused by an improperly grounded system.
- (3)** This unit should only be used as intended by the manufacturer. Any other use must be considered inappropriate and consequently dangerous. The manufacturer will not be held responsible for damage caused by improper, erroneous or unreasonable use.
Do not modify, change or put out of order any safety protection of the equipment. The user must not attempt to carry out any repair unless authorized. Do not clean or repair the **HS-XRV** while it is connected to an electrical system. Regularly check if the power cable is damaged.
When authorized technicians decide to replace spare parts, they must use original replacements and moreover they must adhere to manufacturer's information and technical characteristics.
However use the **HS-XRV** in accordance with the general safety and accident prevention rules of the country in which the equipment is installed.

DESCRIPTION OF THE EQUIPMENT AND ITS USE :

The **HS-XRV** is assembled in accordance with safety rules. The lamps are in accordance with clinical directions that required a very white light (6500° Kelvin) for a precise reading of details. The **HS-XRV** is equipped with one filtered plexiglas screen exempt from fault. The casing is painted with a scratch resistant coating. The unit is wall mountable or can be used on a table top in horizontal, vertical of inclined position using proper support. The **HS-XRV** must be used only for its designed purpose: X-ray films' reading. It is necessary to take all precautions to avoid unauthorized use.

The **HS-XRV** is fragile and is packaged accordingly. Use only the original package if the equipment must be returned to the manufacturer. To carry the **HS-XRV** without package, keep it in horizontal position and use the shortest sides as holding point.

TECHNICAL SPECIFICATIONS :

	HS-XRV 30	HS-XRV 38
Housing	Stainless steel frame	Stainless steel frame
Overall dimensions	w. 33 x h. 20,5 x d. 5,5 cm	w. 41 x h. 25 x d. 7 cm
Screen	30 x 16,5 cm	38 x 20,5 cm
Voltage	230V/50Hz	230V/50Hz
Power	16 Watt	32 Watt
Weight	kg 2,5	kg 3,5
Lamps	2 of 8 Watt	2 of 16 Watt
Light	6500°K daylight	6500°K daylight

IMPORTANT WARNINGS :

- (1) The equipment should only be used by personnel who have read the instructions fully.
- (2) Place the **HS-XRV** on a level surface so that air can freely circulate to avoid any overheating of the lamps. Do not put any objects on the screen.
- (3) Do not put any object on the screen.
- (4) Do not clean or repair the **HS-XRV** while it is connected to an electrical system.

INSTRUCTIONS FOR USE :

- (1) Remove the equipment from the package handling with care.
- (2) Check to see that the main voltage corresponds to the power requirements for the unit as specified on the identification label. Insert cable into Xray viewer and plug into socket.
- (3) Press the main switch on position "I".

MAINTENANCE :

Always check the power is unplugged before maintenance is carried out. The **HS-XRV** needs no particular maintenance, except for a regular cleaning. Clean the equipment with a soft cloth and water - do not use acid or solvents. Regularly check if the power cable is in good condition. It is possible to replace the fuse in the plug, ensure that the same rating fuse is used. If the **HS-XRV** doesn't work properly, contact the manufacturer or the authorized service. If any repairs are needed, please return the equipment directly to the manufacturer or to the local distributor using only the original package. However it is possible to contact specialized technicians, who must work in accordance to general safety and accident prevention rules of the country in which the equipment is installed.

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