hotty

Nr. 1460-0000 / Nr. 1460-1000

Bedienungsanleitung
Instruction manual • Mode d'emploi
Istruzioni per l'uso • Instrucciones de servicio
Инструкция по эксплуатации
取扱説明書 • 사용 설명서

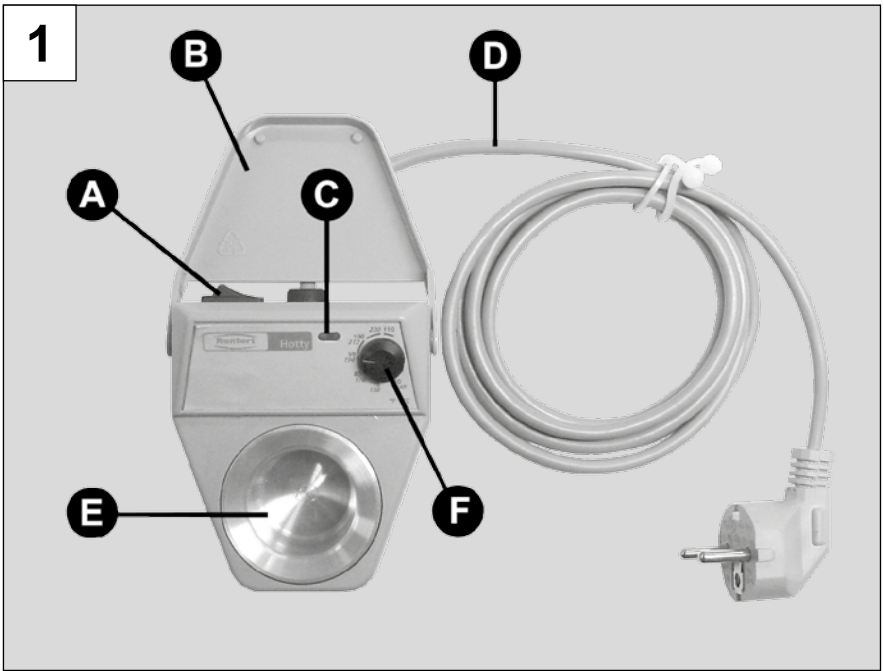
21-9867 01062017

Made in Germany

Renfert

Ideas for dental technology

1. Einleitung

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ENGLISH

1. Introduction

We are pleased with your decision to purchase a *hotty* wax heating unit.

Please observe the following information in order to ensure long and trouble-free service.

2. Application Area

The *hotty* is an electronically regulated wax dipping pot. It is ideally suited for thermal processing with all conventional dipping waxes.

The following special characteristics make the quality of the *hotty* apparent to you during operation:

On the one hand, the **precise temperature regulation** (+ / - 2 °C [3.6 °F]) ensures a uniformly high quality copings and, on the other, working together with the heater that reacts optimally to the temperature conditions, prevents an overheating of the wax and resultant damage.

Please note that the warm-up phase is intended to be gentle on the wax. It therefore may take some time before the wax is completely liquefied. The **continuous increase of the operating temperature** prevents the temperature in the wax pot from exceeding the desired setpoint. The quality of the wax is thus maintained.

The *hotty* offers you **optimal working conditions**. For example, the housing is constructed so that both left- and right-handed individuals can rest their hands on the work while working at the unit. In addition, the dipping pot cover can be swiveled completely out of work area.

2.1 Ambient Conditions

(in accordance with IEC 60664-1)

The *hotty* may only be operated:

- Indoors;
- Up to an altitude of 2,000 m (6,500 ft) above sea level;
- At an ambient temperature of 5 – 40 °C [41 – 104 °F];
- At a maximum relative humidity of 80 % at 31 °C [87.8 °F], decreasing linearly to 50 % relative humidity at 40 °C [104 °F];
- With a main power supply where current fluctuations do not exceed 10 % of the nominal value;
- At pollution degree 2;
- Over voltage category II.

3. Safety Information

- **Before starting the unit, compare the information on the nameplate with the specifications of your local power supply.**
- **Units with damaged power cords or other defects may not be operated.**
- **During operation, always unroll the power cord (D) (overheating hazard in case of a short circuit).**
- **It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE (Association for Electrical & Electronic Technology) 0701-0702.**

- Liquid waxes can cause burns. Do not move the unit when in use!
- During operation, the wax pot (E) is heated up to its rims. Improper use or handling can result in burns even approx. within 30 minutes after power off.
- The *hotty* is designed to be used exclusively in conjunction with commercially available dental waxes. Never use the unit to heat other types of materials!
- Please note:
Overfilled pots can overflow!
- Materials whose flash point is under 175 °C [347 °F] may not be stored in proximity to the *hotty* or come in contact with the *hotty*.
- Do not rinse or dip into water. Hazardous voltage can cause electric shock!

3.1 Liability Exclusion

Renfert GmbH shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions;
- The product is altered in any way other than those alterations described in the operating instructions;
- The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed;
- The product continues to be employed, despite obvious safety faults;
- The product is subjected to mechanical jarring or is dropped.

4. Operating Elements (Fig. 1)

- A) On / Off switch
- B) Lid
- C) LED indicator
 - >> Operation LED = green
 - >> Warm-up phase = red
- D) Power cord
- E) Wax pot
- F) Potentiometer

5. Commissioning / Operation

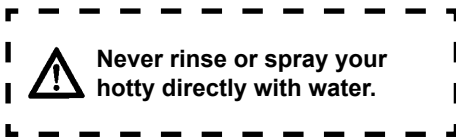
1. Connect the *hotty* to the wall outlet (Fig. 2).
2. Swivel the cover (B) back (Fig. 3).
3. Add the dipping wax to the pot (E) (Fig. 4).
4. Switch the *hotty* on at the main power switch (A).
The operation LED (C - green) then goes on. During the warm-up phase the red LED (C) also remains on.
5. Turn the potentiometer (F) that gives you setpoint between 60 °C [140 °F] and 110 °C [230 °F] in increments of 10 °C [18 °F] until you reach the desired operating temperature (Fig. 5).
6. Once the unit reaches the setpoint temperature, the red LED goes out (C). The red LED would light up shortly when regulating the temperature successively.

5.1 Correct dipping (Fig. 6)

- Support the hand comfortably.
- Do a quick dip to just beyond the preparation border.
- Slowly and evenly remove the die from the wax.
- Just before the tip comes out hold it steady in the pot for a short time so that any excess wax can drip down.

6. Cleaning / Maintenance

The *hotty* is maintenance free. From time-to-time, you should wipe your unit down with a damp cloth. **Never** use detergents or solvents!

With the unit switched off, you can remove hardened wax residue by carefully scraping it.

Warming the unit's surfaces with a hair drier will make it easier to remove wax residue.

7. Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at www.renfert.com/p918.

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

8. Warranty

Provided the unit is properly used, Renfert warrants the hotty for a **period of 3 years**. The original sales invoice of the specialist depot is required for a claim under guarantee. Parts subject to natural wear and tear and also consumables are not covered by the guarantee.

The guarantee is rendered void in the case of incorrect use, non-adherence to the operating, cleaning, servicing or connection instructions, repairs carried out by the owner or repairs that are not carried out by a specialist firm, use of replacement parts from another manufacturer or unusual or unapproved uses not specified in the user guidelines.

Successful claims under guarantee do not extend the guarantee period.

9. Technical data

Maximum power: 25 W (+ / - 10 %)

Minimum operating temperature: 60 °C [140 °F]

Maximum operating temperature: 110 °C [230 °F]

Supply voltage: 220 - 240 V / 50 / 60 Hz
100 - 120 V / 50 / 60 Hz

Fuse protection:

Fuse (not replaceable) / rating:
220 - 240 V = T 500 mA / 250 V
100 - 120 V = T 250 mA / 250 V

Dimensions 90 x 65 x 98 mm
(W x H x D): [3.54 x 2.56 x 3.86 inch]

Weight, approx.: 360 g [12.7 oz]

10. Standard Delivery

1 *hotty* wax heating unit

1 Set of operating instructions

11. Accessories

GEO-Dip dipping waxes:

No. 482-3000 *GEO-Dip* yellow, 200 g

No. 482-3200 *GEO-Dip* orange, 200 g

No. 482-3300 *GEO-Dip* green, 200 g

Rewax dipping waxes:

No. 651-0000 *Rewax* yellow, 210 g

12. Disposal instructions for countries in the EU

To conserve and protect the environment, prevent environmental pollution and improve the recycling of raw materials, the European Commission adopted a directive that requires the manufacturer to accept the return of electrical and electronic units for proper disposal or recycling.

Within the European Union units with this symbol should not therefore be disposed of in unsorted domestic waste:

For more information regarding proper disposal please apply at your local authorities.

13. Troubleshooting Guide

Error	Possible cause	Corrective action
No display / no operation, although the unit is switched on the main switch (A).	<ul style="list-style-type: none">• Power supply interrupted.• Faulty overheat fuse.• Faulty unit fuse.	<ul style="list-style-type: none">• Check the power supply and power cords.• Have fuse replaced by Service Department.• Have fuse replaced by Service Department.
The unit is operating but there is no increase in temperature.	<ul style="list-style-type: none">• Faulty thermal sensor.• Faulty heater.	<ul style="list-style-type: none">• Have probe replaced by Service Department.• Have heater repaired by Service Department.

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Renfert GmbH, Industriegebiet, 78247 Hilzingen / Germany
Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany