
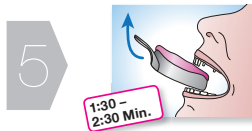
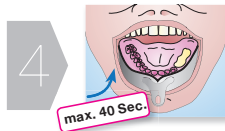


# Visalys® Temp Kurzanleitung

Farbe auswählen, Kartusche einlegen, Verschluss entfernen, neuen Mischer aufsetzen.

Vor jedem Gebrauch etwas Material austragen und verwerfen, Stoppuhr starten.

Vom Boden aus befüllen, Spitze der Mischkanüle im Material belassen. (Tipp: erbsengroße Menge auf den Handrücken austragen, siehe Schritt 5)

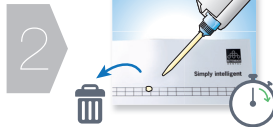

Innerhalb von 40 Sekunden nach Mischbeginn mit leichtem Druck in den Patientenmund einsetzen.

Aushärtegrad intraoral prüfen (oder an der Probe auf der Hand, siehe Schritt 3). Das Provisorium zwischen 1:30 Min. und 2:30 Min. nach Mischbeginn entnehmen.

Ausarbeitung von Visalys® Temp ab ca. 4 Min. nach Mischbeginn.




## Visalys® Temp Abridged instructions


Select shade, insert cartridge, remove cap, attach new mixing tip.




Before each use, extrude and discard a little material, start a stopwatch.

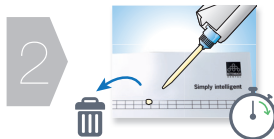

Fill from the base upwards, keep the end of the mixing tip in the material. (Tip: apply a pea-sized quantity of material to the back of the hand, see step 5)

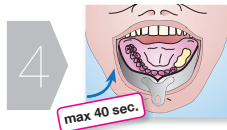

Insert into the patient's mouth, applying slight pressure, within 40 sec. of starting the mixing.

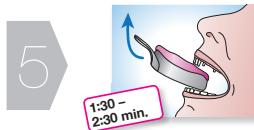

Check the degree of hardness intra-orally (or use the sample on your hand, see step 3). Remove the temporary restoration between 1:30 min. and 2:30 min. after the start of mixing.



Finish the Visalys® Temp about 4 min. after the start of mixing.

