

Eurosept® Plus Evac Disinfection Concentrate

- ⒺS Desinfectante concentrado para sistemas de evacuacion
- ⒻR Désinfectant concentré pour systèmes d'aspiration
- ⒹE HS-Absauganlagen-desinfektion, Konzentrat
- ⒾT Disinfettante concentrato per gli impianti di aspirazione
- ⒶL Evac desinfectieconcentraat
- ⒶT Concentrado de desinfeccão Evac
- ⒸS Dezinfeční koncentrát Evac
- ⒶL Koncentrat do dezynfekcji Evac



EN: Areas of application: High performance solution designed for cleaning and disinfecting of dental Evacuation systems (incl. suction hoses and amalgam separator) and spittoons. **Instructions for use:** The aldehyde-free concentrate must be first diluted with water. Recommended application concentration: 2%. Mix 20 ml concentrate with 980 ml of tap water and aspirate the solution through the suction hoses. Regularly, the suction system should be disinfected once or twice a day. The spittoon is disinfected by puring approx. 250 ml of the ready-to-use dilution. Do not mix with other disinfectants or cleansers! Always prepare ready-to-use solutions fresh and use immediately. **Effectivity:** Bacteria and *Candida albicans* in accordance to WHI (high contamination): 2,0 % - 5 min.; 1,0 % - 60 min.; Bacteria in accordance to EN 13727 and EN 14561 (high contamination): 3,0 % - 2 min.; Yeast/acid (Candida albicans) in accordance to EN 13624 and EN 14562 (high contamination): 3,0 % - 2 min.; Limited virucidal (Enveloped viruses) i.e. HBV, HIV, HDV) in accordance to BGR (RN)DVG (contamination): 2,0 % - 2 min. Safety guidelines: Causes severe skin burns and eye damage. Toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/ eye protection. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/ attention. If eye irritation persists: Get medical advice/ attention. Dispose of contents/ container in accordance with all local, regional, national and international regulations. Contains N,N-Dimethyl-N-Dioctylammoniumchlorid, Kaliumhydroxid. **Directive 648/ 2004/ EC:** Contains < 5% nonionic surfactants, < 5% cationic surfactants, fragrances.



see imprint

LOT

Data Matrix
Code



UN 1903

Danger
Peligro
Danger
Gefahr
Pericolo
Gevaar
Perigo

Niebezpieczeństwo

Niebezpieczeństwo

5 L e

Medical Device
Dispositivo médico
Dispositiv médical
Medizinprodukt
Dispositivo medicale
Medisch hulpmiddel
Dispositivo médico
Zdravotnických prostředků
Wyrób Medyczny

40 °C
104 °F
5 °C
41 °F

CE 0482

Ⓕ HENRY SCHEIN INC.
135 DURYEA ROAD
Melville, NY 11747 USA

[CE REF]

HENRY SCHEIN SERVICES GmbH
MONZSTRASSE 2A
D-63225 LANGEN, GERMANY
TEL: +49 6103 757 5000
WWW.HENRYSCHEINBRAND.COM
Rev.2019/07

