B.A. ULTiseal BASE740

Heatsealer

Heatsealer

Thermosoudeuse

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Termoselladora

Termosigillatrice

Thermosealing

REF BA140600

Instructions for use
Manuel d'utilisation
Benutzerhandbuch
Manual de usuario
Istruzioni per l'uso
Instructies voor gebruik

 **B.A. International Ltd.**
Unit 9, Kingsthorpe Business
Centre Studland Road,
Northampton, NN2 6NE U.K.

EC **REP**

CB Healthcare Consulting GmbH
Am Neumarkt 34
22041 Hamburg
Germany

Dear Doctor,

*Thank you for the faith that you have given us by acquiring our thermosealing machine **ULTiseal BASE740**.*

We wish to inform you that our firm is at your complete disposition to provide information and to answer any questions concerning the operation and use of this equipment.

*We remind you that for the correct usage of this machine it is necessary to first read this manual. The **ULTiseal BASE740** corresponds to all of the relative safety regulations and presents no danger to the operator when used according to the instructions. Please also note that B.A. International declines all liability for incorrect or insufficient interpretations of the translations of this manual: in the event of a dispute, only the manual written in the English language shall apply.*

While wishing you success in your work, we remind you that the reproduction of this manual is forbidden and that the technical characteristics could be changed without warning due to our continued technical research.

B.A. International Ltd.

INDEX

INTRODUCTION	3
GENERAL NOTES ON DELIVERY	3
GENERAL INSTRUCTIONS	3
EXPLANATION OF SYMBOLS.....	4
GUARANTEE INFORMATION.....	4
CHAPTER 1	5
1.1 - DIMENSIONS OF THE MACHINE AND OF THE PACKAGE (in mm).....	5
CHAPTER 2	6
2.1 - INSTALLATION.....	6
2.1.1 - Instructions for wall mounting	6
CHAPTER 3	7
3.1 - SAFETY	7
3.2 - ELECTRICAL CONNECTION.....	8
CHAPTER 4	9
4.1 - DESCRIPTION.....	9
CHAPTER 5	10
5.1 - INSTRUCTIONS FOR USE	10
CHAPTER 6	12
6.1 - CLEANING INSTRUCTIONS.....	12
6.1.1 - Special maintenance and warnings	12
CHAPTER 7	13
7.1 - PROBLEM SOLVING GUIDE	13
CHAPTER 8	14
8.1 - DISPOSAL INSTRUCTIONS	14

INTRODUCTION

GENERAL NOTES ON DELIVERY

Upon receipt of the equipment, please check the condition of the package and keep it for possible shipments.

Open the package and check that:

- the equipment corresponds to the specific requirements;
- there is no evident damage.

In case of damage or missing parts, inform immediately either the shipping agent, the dental depot or B.A. International.

All of the dimensions listed in this manual are approximate.

The sketches and any other consigned documents regarding this machine are copyrighted. They cannot be made available to any third parties.

It is also forbidden to reproduce in whole or in parts either the instruction text and the illustrations.

GENERAL INSTRUCTIONS

Included in this manual are instructions for:

- correct installation;
- the safe and efficient use of the apparatus;
- routine maintenance.

The unit must be used in compliance with the procedures contained in the manual and never for reasons other than those specified therein.

The **operator** is the person who physically uses the apparatus for the purposes for which it has been designed.

The **responsible authority** is the person or group responsible for its use, for the ordinary maintenance of the apparatus and for the training of the operator.

The responsible authority is to be held legally responsible for the performance of the unit as far as it is a direct result of its installation, operation and use.

The manufacturer cannot be held responsible for any breakage, damage or malfunctioning of the unit if the machine has not been used correctly or not adequately maintained.

Conditions of use: this manual is an integral part of the product and must be kept near the unit in order for it to be easily and quickly consulted. The unit is for professional use only and only trained persons may use it. The unit is to be used only for the purpose for which it was designed.

ULTiseal BASE740 is a machine for thermosealing and cutting of sterilisation rolls made of polypropylene paper.

EXPLANATION OF SYMBOLS

Safety symbols found on the equipment, labelling, packaging and in the IFU: These symbols are intended to draw the attention of the reader to a particular note or explanation. Their meanings are explained below.

SYMBOL	DESCRIPTION
	WARNING: HIGH TEMPERATURE
SN	SERIAL NUMBER The symbol must be accompanied by the manufacturer's serial number. The serial number must be adjacent to the symbol.
	PRODUCTION DATE The symbol must be accompanied by the year. The year must be composed of 4 digits.
	WARNING, READ THE INSTRUCTION MANUAL
	MANUFACTURER
	AUTHORIZED REPRESENTATIVE IN THE EUROPEAN UNION
	CE MARK
	ENVIRONMENT PROTECTION Should not be disposed of with other household waste.
	IMPORTANT SAFETY INFORMATION This symbol is used to draw the attention of the reader to notes of particular importance for the safety of the user.

GUARANTEE INFORMATION

B.A. International guarantees the quality of its equipment, if used in accordance with the instructions supplied in this manual.


The heatsealer ULTiseal BASE740 has a warranty of 12 months from date of purchase.

In the case of dispute, the date indicated on the purchase invoice, showing the serial number of the unit, will be considered as valid.


NOTE: please retain the original packaging and use it each time the unit needs to be transported. The use of different packaging could damage the product during transportation.

CHAPTER 1

1.1 - DIMENSIONS OF THE MACHINE AND OF THE PACKAGE (in mm)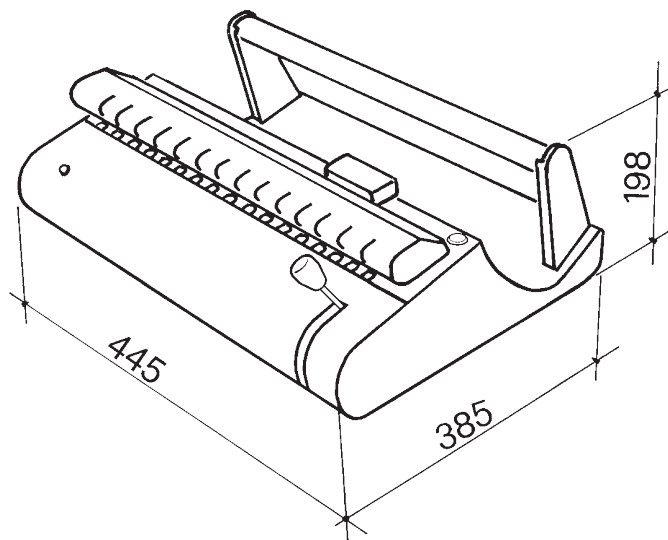

Dimensions of the ULTiseal BASE740 electronic thermosealing machine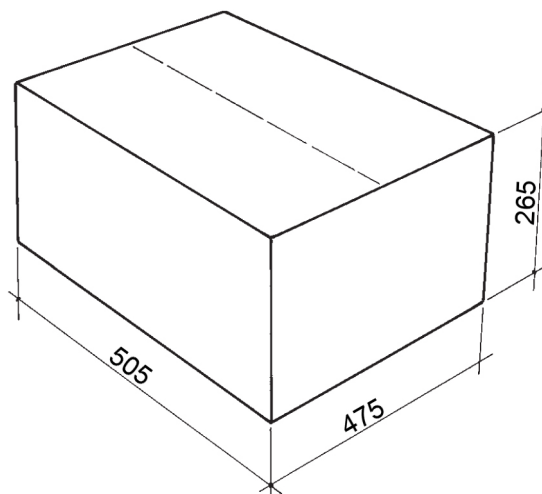

Overall dimension of the package

CHAPTER 2


2.1 - INSTALLATION

The thermosealing machine ULTiseal BASE740 has been tried and tested in the factory.
Remove the packing and install the apparatus using the following instructions:

1. Install the machine on a smooth flat surface with a remaining space of a least 50-80 mm. on all sides.
2. Do not install the machine close to a sink or similar area to avoid contact with water and possible electrical short circuits.
3. Keep away from heat sources
4. Install the machine in a ventilated area.
5. Ensure that the electrical cable is not folded up and freely reaches the electrical socket.
6. The unit must be placed so that the plug is accessible.
7. Insert the plug and the machine is ready for use.

2.1.1 - Instructions for wall mounting

1. Unit does not come with wall mounting screws or wall hangers, please use fixtures suitable for the wall.
2. Please mount in a safe place away from electrical wiring or plumbing in the wall.
3. Follow the directions in the image below to mount the unit to the wall.

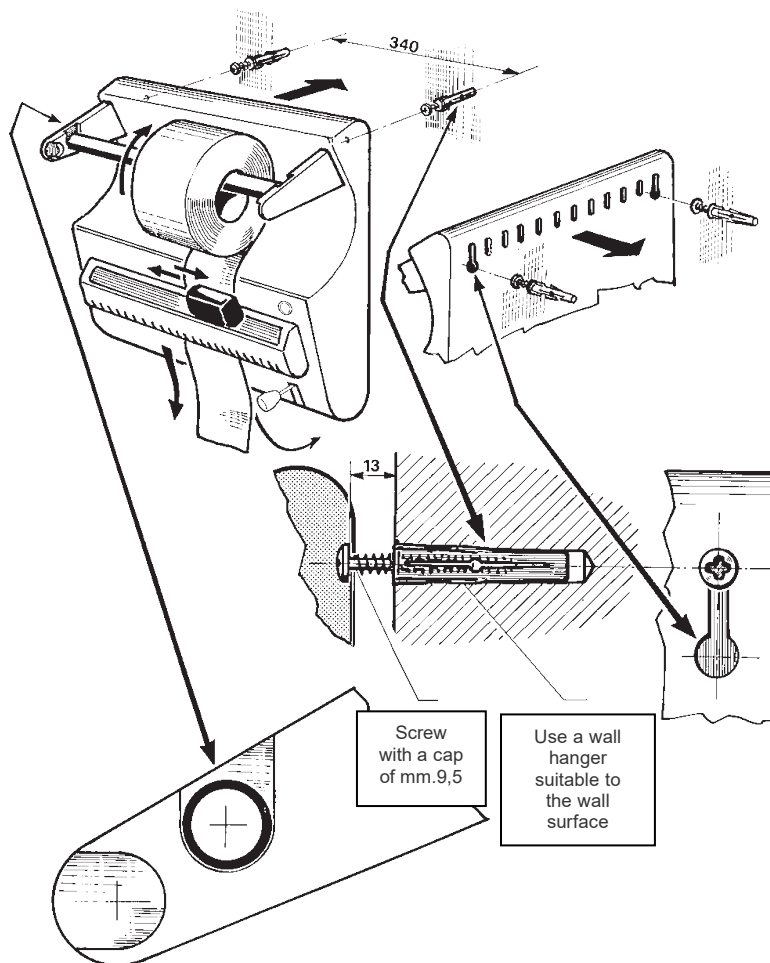CHAPTER 3

3.1 - SAFETY

Before using your electronic thermosealing machine **ULTiseal BASE740**, please read this safety information carefully. Your lack of observance could cause accidents or damage to the machine.

- Before using the unit, operators must have perfectly understood the meanings and functions of all the controls.
- Operators must be aware of and know how to apply the safety regulations governing the use of the unit.
- Operators must know and correctly interpret all the indications contained in this manual and those applied to the unit.
- Operators must not perform operations on their own initiative or operations that are not part of their job.
- The responsible authority must instruct and train the operator to use and maintain the unit safely; in particular, it must ensure that this information has been correctly understood.

NEVER PLACE YOUR FINGERS NEAR THE SEALING AREA WHEN THE MACHINE IS OPERATING
NEVER PLACE YOUR HANDS UNDER THE SEALER WHEN YOU PRESS THE SEALING LEVER
NEVER PLACE YOUR HANDS NEAR THE SHARP EDGE OF THE CUTTING BLADE.

KEEP THE MACHINE IN A CLEAN AND DRY AREA.

NEVER DISMANTLE THE ULTiseal BASE740. THE HIGH INTERNAL VOLTAGE IS DANGEROUS

CLEAN THE MACHINE WITH A DAMP CLOTH, AFTER ENSURING THAT THE CONNECTION CABLE HAS BEEN UNPLUGGED (before reusing the machine make sure that the machine is completely dry). DO NOT USE SOLVENTS ON THE MACHINE LABEL.

MAKE SURE THAT THE WIRING HAS A GROUND CABLE WHICH CONFORMS TO ALL OF THE SAFETY NORMS OF THE COUNTRY OF INSTALLATION.

NEVER REMOVE THE LABEL ON THE MACHINE

ENSURE THAT THE MACHINE IS CONNECTED TO THE CORRECT ELECTRICAL VOLTAGE

If the machine doesn't work properly refer to the section in the manual titled "Solution to Problems" for the possible cause. If you need further information or if the machine is in need of repair, contact B.A. International or an approved dealer.

WARNING

- The machine conforms to the safety regulations established by the Standards Institute and has a two prong plug (EU) or three prong plug (UK) which ensures that the apparatus is shock resistant.
- The device has a double isolation. Only the resistance is connected to earth because the rest of the appliance is in double insulation.
- The most fundamental safety requirements are: a sturdy base and an electrical voltage capacity which is written on the label. In case of doubt have it verified by qualified personnel.

SHOULD THE REQUIREMENTS LISTED NOT BE OBSERVED, B.A. INTERNATIONAL LTD. FORFEITS ALL RESPONSIBILITY FOR THE MISUSE OF THE MACHINE.

3.2 - ELECTRICAL CONNECTION

Check that the electrical current indicated on the label on the rear of the machine corresponds to the voltage of the place of installation.

The machine must be connected to an electrical system with adequate voltage which corresponds to all standards laws of the country of installation.

ATTENTION: Don't fold the electrical cable and don't lean objects on it.

CHAPTER 4

4.1 – DESCRIPTION

ULTiseal BASE740 is the newest electronic thermosealing machine from B.A International, with a continuous cycle and constant temperature.

ULTiseal BASE740 is a machine with a modern, innovative and ergonomic design, easy to use and to clean, with rounded contours and fitted with a roll holder which has an overall dimension that measures the length of the machine. The sterilisation rolls of paper/polypropylene are positioned inside the machine. The new applied technology minimises the energy consumption by creating shorter sealing times and eliminating the need of cooling times between every use. It also increases the reliability of the machine, excluding the periodical servicing and repairs. ULTiseal BASE740 is built in accordance with the quality standards set by the European Community.

CHARACTERISTIC OF THE THERMOSEALER

Supply voltage	230 V
Number of phases	1
Supply frequency	50/60 Hz
Power input	100 W
Isolation class	1/ IPX0
Fuses	N° 2 – 5 x 20 – T5 A 250V
Range of environmental conditions in which the unit was designed to operate	- Indoor use - Altitude up to 2000 m - Temperature: +5 ÷ +40°C - Max relative humidity 85% - Max variation in mains voltage: ±10% - Installation category (overvoltage category) II - Pollution level 2
Switch	Single-pole
Weight of the machine	7.6 kg
Sealing band	12 mm
Type of sealing	constant temperature
Sealing area width	310 mm.
Power unit	unique and autonomous, available for use with 220V which is not subject to short circuiting. It is easy to have repaired (in the rare case of breakage), and is built in compliance with EC safety standards.
Roll hanger	has a hidden support
Cooling time	not needed, between one sealing and another
Positioning of the tape during the sealing	During the sealing there is no difference between placing the sealing tape with the film above or below
Cutting edge	practical, safe, functional and easy to use
Sealing lever	ergonomic, handle with a light and delicate movement
Design	ergonomic and rounded
Colour	RAL 9036, powder painted
Routine maintenance	not necessary


CHAPTER 5

5.1 - INSTRUCTIONS FOR USE

1. Connect the machine to the electricity source using the electrical cord.


Press the lateral switch POWER - **A** -, located on the top of the unit.

The light - **B** -, found at the front of the machine will turn green when the optimal sealing temperature has been reached.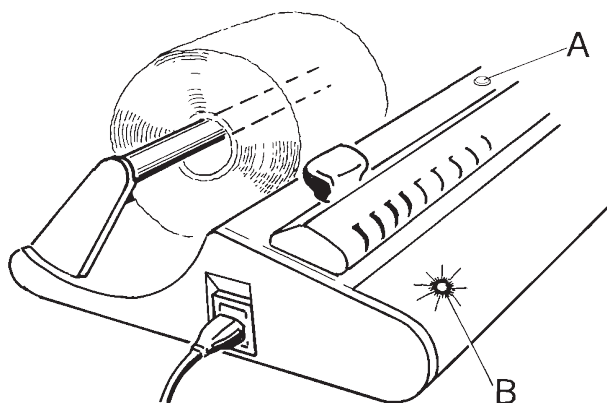


2. Fit the sterilisation roll under the holder -D-.

The roll can be inserted with the film facing up or down.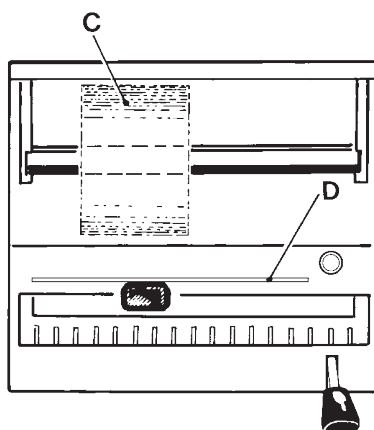

3. Lower the sealing lever. The lever will pass from position -G- to position -H-.

When the sealing procedure is completed the light - I - will turn red and the signal bell will ring.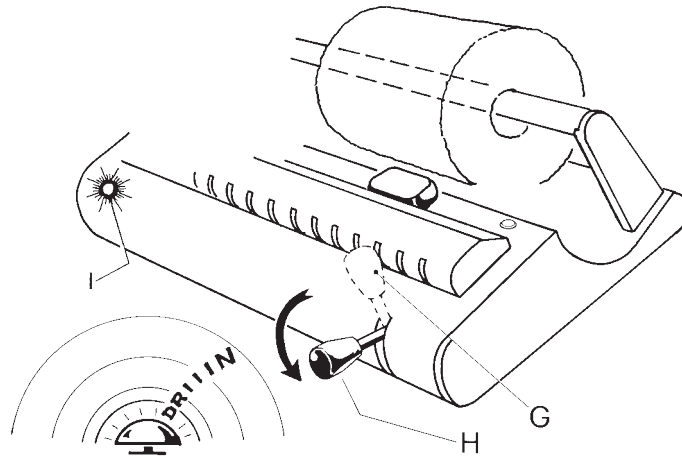


4. Raise the sealing lever passing from position -L- to position -M-.

When the lever is raised the light will turn green again and the signal bell will stop ringing.

The machine is ready for the next sealing without having to wait for a cooling period.

If the operator doesn't return the lever to its start position immediately after the sealing procedure is completed, the power unit will stop itself automatically (so the tape does not burn). The bell however will continue to ring. As soon as the lever is returned to its start position - G - the power unit will automatically switch on again. It will take several seconds to restore the optimal sealing temperature.

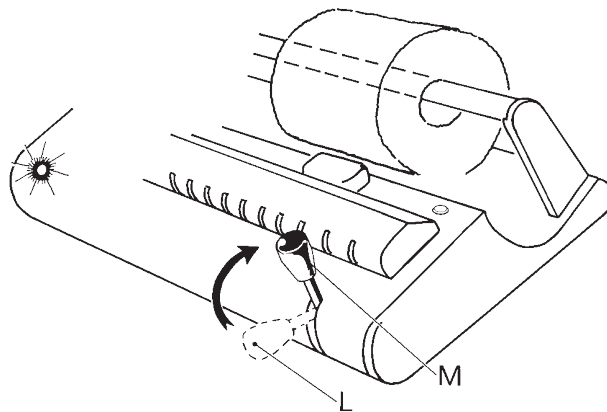CHAPTER 6

6.1 - CLEANING INSTRUCTIONS

Before performing any maintenance operations, carefully read the following safety instructions and, especially, chapter 3 “Safety”.

DANGER: HIGH INTERNAL VOLTAGE.

WARNING: DISCONNECT THE POWER SUPPLY BEFORE STARTING CLEANING.
Non-observance may cause serious injury to people and seriously damage the unit.

ALL MAINTENANCE OPERATIONS MAY ONLY BE PERFORMED BY THE RESPONSIBLE AUTHORITY OR BY THE TECHNICIANS AUTHORISED BY THE ASSISTANCE SERVICE OF B.A. INTERNATIONAL LTD.

- Clean the exterior of the machine with a damp cloth, after making certain that the machine is completely dry. Before starting to work make sure that the machine is completely dry.
During the cleaning of the machine pay attention to the cutting edge position.

Before starting work, make sure the unit is dry. Beware of the knife blade when cleaning the unit.

6.1.1 - Special maintenance and warnings

- The machine does not require routine servicing.
-
- All repair work must be carried out by an approved service centre.

CHAPTER 7

7.1 - PROBLEM SOLVING GUIDE

PROBLEM	CAUSES	SOLUTION
The switch light remains off.	No voltage/electrical current	Check electrical supply
	Protection fuses on the side of the cable	Check for possible short circuit or breakage, if so replace with fuses of the same value
	Switch light broken	Replace the switch fuses set
The panel light doesn't turn on after the machine has been on for several minutes	Panel broken	There may be a fault with the electronic board. Send the unit to BA or an approved service centre for repair.
	The light doesn't function	There may be a fault with the light. Send the unit to BA or an approved service centre for repair.
	The power unit is not heating	There may be a fault with the power unit. Send the unit to BA or an approved service centre for repair.
The sealing process occurs normally, but the light remains green	Micro Switch broken	Send the unit to BA or an approved service centre for repair.
The unit heats, but sealing process is not sufficient	Sealing pressure is not sufficient	Check the screws which support the silicon rubber piece.
	Silicon rubber ruined	Send the unit to BA or an approved service centre for repair.

DANGER: HIGH INTERNAL VOLTAGE. DO NOT DISASSEMBLE.

CHAPTER 8

8.1 - DISPOSAL INSTRUCTIONS

The thermosealing machine ULTiseal BASE740 has been manufactured using iron materials, electronic components and plastic materials. No particular precautions are required to throw the unit away. Do not dump the unit in an unguarded place; consign it to a waste disposal agency for scrapping. Always refer to the specific laws in the country of use when scrapping equipment.

The  symbol on the appliance means that it must be disposed of as “sorted waste”.

The user must therefore send (or instruct other people to send) the unit to one of the sorted waste collection centres set up by the local councils, or send it to the dealer against the purchase of an equivalent unit (European Union and United Kingdom only).

Sorted waste collection and the subsequent treatment, recovery and disposal operations facilitate the production of equipment using recycled materials and limit the negative environmental and health effects that may be caused by improper waste management.

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